beurer MG 150

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Explanation of symbols

The following symbols are used in the operating instructions and on the type plate:

WARNING Warning of risks of injury or health hazards.

CAUTION Safety information about possible damage to appliance/accessories.

Note Important information.

The device has double protective insulation and therefore complies with protection class 2.

Only use indoors.

1. Getting to know your appliance

Using this electric massager, you can give yourself with a soothing, effective massage with no help required. Massages can be relaxing or stimulating and are effective against strained muscles, aches and fatigue. This product provides a powerful, intensive massage for the neck and shoulder area.

The device is for **private use only and is not intended** for medical or commercial purposes.

2. Proper use

The product is exclusively designed for massaging the neck and shoulder area of the human body. It cannot replace medical treatment. Do not use the massage device if any of the following warnings apply to you. Please consult your doctor if you are unsure whether or not the massage device is suitable for you.

To avoid damage to health, we strongly advise against using the massager in the following cases:

- With implanted electrical devices (such as a pacemaker)
- In the case of metal implants
- On children and animals
- If you suffer from a medical abnormality or injury in the area to be massaged (e.g. herniated disc, open wounds)
- On the head and on hard parts of the body (e.g. knees and joints)
- On the face (eyes), on the larynx or any other particularly sensitive body parts
- On swollen, burnt, inflamed or damaged skin and body parts
- On cuts, capillaries, varicose veins, acne, couperose, herpes or other skin diseases
- During pregnancy
- Whilst sleeping
- After the consumption of medicines or alcohol (reduced cognition!)
- In vehicles.

This device is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking the required understanding of the equipment and how it is used, unless they are supervised by a person who is responsible for their safety or they are given instructions on how to use the device.

Children should be supervised to ensure that they do not use the device as a toy.

You should consult your doctor before using the massage device if:

- you suffer from a serious illness or are recovering from an operation on the upper body,
- Thromboses,
- Diabetes,

- in case of pains of unclarified origin.

The device is only intended for the purpose described in these operating instructions. The manufacturer cannot be held liable for damage occurring as a result of inappropriate or careless use.

3. Safety information

Please read these operating instructions carefully. Failure to observe the information provided below can lead to personal injury or material damage. Keep these instructions in a safe place where they can also be read by other persons using the device. Always pass on these instructions together with the appliance if you sell it or pass it on.

WARNING

Keep children away from the packaging materials. Risk of suffocation.

Electric shock

WARNING

Just like any other electrical appliance, this massage device must be handled with due care and attention in order to prevent the risk of electric shock.

For this reason, the device must only be operated as follows:

- with the mains voltage specified on the device only,
- do not use the device if damage to the device itself or any of its accessories is evident,
- never use the appliance during a thunderstorm.

In the event of a defect or malfunction, switch the device off immediately and disconnect it from the power supply. Never pull on the power cable or on the appliance in order to disconnect the plug from the mains outlet. Never hold or carry the device by the power cable. Keep the cables away from hot surfaces. Do not pierce the cable or device with needles or pointed objects.

Ensure that the massager and the mains cable do not come into contact with water, steam or other liquids. Therefore, use the device only in dry interior rooms (e.g. never in the bath, sauna).

Never try to retrieve the device if it falls into water. Disconnect the plug immediately from the mains outlet. Do not use the device if there is visible damage or if fluid is leaking from the device. Do not subject the device to any impact and do not drop it.

Repairs

WARNING

- Repairs to electrical appliances must only be carried out by qualified persons. Incorrect repairs can lead to considerable danger for the user. In the event of repairs, please contact our customer service or an authorized dealer.
- The zip fastener on the massager must not be opened. It is only used for technical production reasons.
- Please have an authorised retailer replace a damaged lead.

Fire hazard

WARNING

There is a risk of fire if the device is used incorrectly or if it is not used according to these operating instructions.

For this reason, the massage device must only be operated as follows:

- Never use the device under covers, such as blankets or pillows.
- Never use the device in the vicinity of petrol or other easily flammable materials.

Handling

CAUTION

- Switch the device off and disconnect the power supply after every use and before every cleaning.
- Do not insert any objects into the device. Ensure that the moving parts can move freely at all times.
- Never sit or stand on the moving parts of the device with all your weight and do not place objects on the device.
- Please have an authorised retailer replace a damaged lead.

Disposal

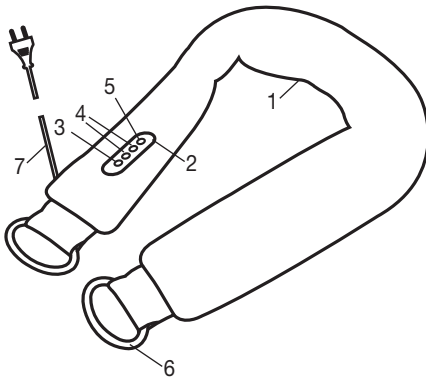
CAUTION

Follow local regulations regarding the disposal of materials. The appliance should be disposed of according to Regulation 2002/96/EC-WEEE (Waste Electrical and Electronic Equipment). In case of queries, contact the municipal authorities responsible for waste disposal in your area.

4. Appliance description

1. Massage heads (on the underside of the device)
2. Operating element
3. Button  with LED indicator
4. +/- buttons
5. Mode button
6. Handles
7. Mains cable

5. Start-up

- Remove the packaging
- Check the device and cable for damage.
- Route the cable safely so that there is no risk of anyone tripping over it.

6. Operation

The massage must be pleasant and relaxing at all times.

Switch on the massager using the  button. The massage is always started on the lowest setting and in the first massage programme. Three massage intensity settings can be selected using the +/- buttons. The device offers six different massage programmes that can be selected by pressing the Mode button. Switching the device off resets the default settings, meaning that the device will start in the first massage programme and the lowest intensity when it is switched on the next time.

Using the handles, the contact pressure can be increased or decreased and the position of the device can be adjusted. Press the  button again to turn off the device.

Note

- Do not use the massage device for longer than 15 minutes. Otherwise, using the device for longer can over-stimulate the muscles and cause tension instead of relaxation.
- The device has an automatic shut-off mechanism set to a maximum operating period of 15 minutes.
- Allow the device to cool for 15 minutes before using it again.
- The massager features protection from overheating with an automatic switch-off fixture if the device overheats. In such cases, let the device cool down for 30 minutes before recommencing use.

7. Care and storage

CLEANING

WARNING

- Disconnect the device before cleaning it.
- Clean the device using only the methods specified. Under no circumstances may liquid enter the device.
- Small marks can be removed using a damp cloth. Please only clean the top side of the device.
- Do not use the device again until it is completely dry.

Storage

If you do not use the device for longer periods, we recommend storing it in the original packaging in a dry place without weighting it down.

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