<b>DE</b>	Massageball mit Vibration Gebrauchsanweisung.....	2
<b>EN</b>	Vibrating Massage Ball Instructions for use.....	20
<b>FR</b>	Balle de massage vibrante Mode d'emploi .....	37
<b>ES</b>	Bola de masaje con vibración Instrucciones de uso .....	55
<b>IT</b>	Pallina per massaggi con vibrazione Istruzioni per l'uso .....	72
<b>TR</b>	Titreşimli masaj topu Kullanım kılavuzu .....	89
<b>RU</b>	Массажный мяч с вибрацией Инструкция по применению ...	105
<b>PL</b>	Piłka do masażu z wibracjami Instrukcja obsługi .....	122

# DEUTSCH

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## ENGLISH

**Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.**

### **WARNING**

- The device is intended only for domestic/private use, not for commercial use.
- This device can be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely and are aware of the consequent risks of use.

- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- Under no circumstances should you open or repair the device yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to comply will invalidate the warranty.

**Dear customer,**

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air.

With kind regards,  
Your Beurer team

**Table of contents**

1. Getting to know your device .....	23
2. Included in delivery.....	23
3. Signs and symbols .....	23
4. Warnings and safety notes .....	25
5. Intended use.....	27
6. Device description.....	29
7. Initial use.....	29
8. Usage .....	30
9. Cleaning and maintenance .....	33
10. Disposal.....	33
11. Technical specifications .....	34
12. Warranty/service.....	35

# 1. Getting to know your device

The MG 10 Vibrating Massage Ball allows you to carry out a targeted spot massage of tense muscle groups. It can also help to loosen fascial tissue and break down fasciae agglutinations. The massage ball is comfortable to use thanks to its soft-touch surface.

It can be operated at 2 different intensity levels, meaning that gentle or strong vibration provides different massage intensities, depending on your personal preference.

# 2. Included in delivery

1 x Vibrating Massage Ball

3 x 1.5 V batteries (AAA)

1 x Set of these instructions for use

# 3. Signs and symbols

The following symbols appear in the instructions for use and on the device:

	<b>WARNING</b>	Warning notice indicating a risk of injury or damage to health.
	<b>IMPORTANT</b>	Safety note indicating possible damage to the device/accessory.
	<b>Note</b>	Note on important information.

	<p>Read the instructions for use</p>
	<p><b>CE labelling</b> This product satisfies the requirements of the applicable European and national directives.</p>
	<p>Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE</p>
	<p>Do not dispose of batteries containing hazardous substances with household waste</p>
	<p>Manufacturer</p>
	<p>Dispose of packaging in an environmentally friendly manner</p>
	<p>The products demonstrably meet the requirements of the Technical Regulations of the EAEU.</p>
	<p>Direct current</p>

## 4. Warnings and safety notes

Read these instructions for use carefully! Non-observance of the following information may result in personal injury or material damage. Store these instructions for use and make them accessible to other users. Make sure you include these instructions for use when handing over the device to third parties.

### **WARNING**

- Keep packaging material away from children. There is the risk of suffocation.
- Children must be supervised at all times whilst using the device.

There is a risk of fire if the device is used improperly or these instructions for use are ignored.

Therefore, the device must

- Never be left lying around unattended, especially if there are children nearby
- Never be used underneath a cover, such as a blanket, pillow, etc.
- Never be used near petrol or other highly flammable substances.
- Only be used in dry interior rooms (e.g. never in the bath or sauna)

Make sure that the device does not come into contact with water or other liquids.

Never reach for a device that is submerged in water.

## Important

- Before use, ensure that the device and accessories display no visible damage. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.
- Never open the housing.
- Keep the device away from pointed or sharp objects.
- If the device has been dropped or has suffered any other damage, it must no longer be used.
- The device must be switched off after every use and before cleaning.

## Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, rinse the affected area with water and seek medical assistance.
-  **Choking hazard!** Small children may swallow and choke on batteries. Therefore, store batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
-  **Risk of explosion!** Do not throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.

- Do not use rechargeable batteries.
- Do not disassemble, open or crush the batteries.

## Repairs

### **WARNING**

- Only specialist personnel may perform repairs on electrical devices. Improper repairs may subject users to considerable danger. For repairs, please contact Customer Services or an authorised retailer.

## 5. Intended use

This device is intended to massage individual parts of the human body. It cannot replace medical treatment. Do not use the massager if one or several of the following warnings applies to you. If you are unsure whether the massager is suitable for you, please consult your doctor.

### **WARNING**

Do not use the massager:

- On animals
- If you suffer from a medical abnormality or injury in the area to be massaged (e.g. herniated disc, open wounds),
- Under any circumstances for massages in the area around the heart if you wear a pacemaker. If this is the case, consult your doctor regarding other parts of the body first
- On swollen, burnt, inflamed or damaged skin and body parts of any kind

- On cuts, capillaries, varicose veins, acne, couperose, herpes or other skin diseases
- During pregnancy
- On the face (eyes), on the larynx or other particularly sensitive body parts
- While sleeping
- After the consumption of medicines or alcohol (reduced cognition!)
- In a vehicle
- In the pubic area

You should consult your doctor before using the massager, especially

- If you suffer from a severe illness
- If you have a pacemaker, implant or other aids
- If you suffer from thrombosis
- If you suffer from diabetes
- if you are from suffering pain, the cause of which is unknown.

The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.


## 6. Device description

On/Off button  
(setting of the 2 intensity levels)

## 7. Initial use

- Remove the packaging.
- Check the device and batteries for damage.
- Open the massage ball by twisting in opposite directions with your hands.
- Place the batteries in the battery compartment properly.
- Put the two halves of the ball back together and twist them again until the two halves click into place. The two imprinted arrows on the ball are now at the same level again.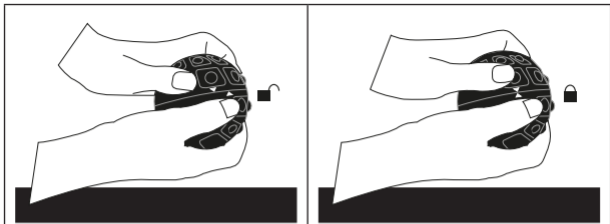

## 8. Usage

The massage should feel pleasant and relaxing at all times. If the massage is painful or uncomfortable, stop the massage or change your position or the contact pressure.

- You can switch on the device by pressing the On/Off button once. The massage ball starts to vibrate at the strongest intensity level.
- If you press the On/Off button again, the ball starts to vibrate at the weaker of the two intensity levels. Select one of the two intensity levels depending on your personal preferences.


### **i** Note

- We recommend treating each area (lower body, mid-body and upper body) three times a week for 15 minutes each time. When carrying out a session, we recommend training all three areas. However, you may also choose to focus on one of the above areas. Carry out treatment consciously and slowly.

## Examples of use:


### Calf

Sit on the floor and place one leg on the massage ball so that it is positioned on the bottom half of the calf. Support yourself with your hands while leaning back and lift your hips. Tense your stomach and bottom muscles and shift your lower body forwards and backwards. The calf muscles are stimulated.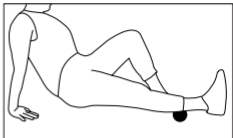


### Thigh/bottom

Lie on your side on the floor and stretch both legs straight out. Bend one arm and support yourself on the floor with your elbow while keeping your other arm relaxed by your side. Place the massage ball under your hips. Now roll over the ball with slow movements. Vary the exercise by supporting yourself with both arms while leaning back; then roll your bottom over the ball with gentle pressing movements.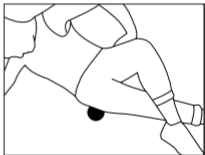

### Chest


Sit upright on the floor or on a chair and hold the ball in one hand. Now roll it over your chest in circular movements. In order to increase the pressure, you can also lean against a wall while standing up, with your front facing the wall.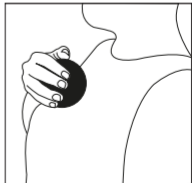

### **Arm/shoulder**


Lean against a wall while standing up, with your side facing the wall. Place the ball between your upper arm and the wall. Now apply gentle pressure on the ball and move up and down with your legs so that the ball rolls slowly along your arm. Increase the distance between your legs and the wall in order to increase the pressure on the ball.

Now slowly turn your back to the wall so that the ball moves along your shoulder area and the muscles in your shoulders are stimulated.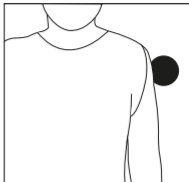

### **Back**

Turn your back to the wall and place the ball on the right or left next to the spine between your back and the wall. Now move slowly up and down and roll the ball over your back.

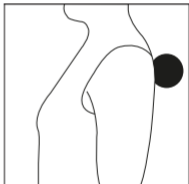### **i Note**

- Please make sure that the ball is not rolled directly on the bone.

## 9. Cleaning and maintenance

### Cleaning

#### **WARNING**

- Switch off the device each time before cleaning.
- Clean the device with a slightly damp cloth only.
- Clean the device using only the methods specified. Under no circumstances may liquid enter the device or accessories.
- Do not use the device again until it is completely dry.
- Do not clean the device in a dishwasher.
- Do not use any abrasive cleaning products or hard brushes.

### Care

If you do not plan to use the device for a long period of time, we recommend that you store it in the original packaging in a dry environment, ensuring it is not weighed down by additional items.

## 10. Disposal

#### **IMPORTANT**

- Batteries must not be disposed of in the household waste. They may contain poisonous heavy metals and are subject to special refuse treatment.

- The codes below are printed on batteries containing harmful substances

Pb = Battery contains lead

Cd = Battery contains cadmium

Hg = Battery contains mercury

For environmental reasons, do not dispose of the device in the household waste at the end of its service life.

Dispose of the device at a suitable local collection or recycling point in your country.

Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

## 11. Technical specifications

Weight	Approx. 162 g
Battery	3 x 1.5 V — — — AAA (LR03) battery
Device dimensions	Diameter approx. 8 cm

## 12. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

**The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.**

**The warranty shall apply without prejudice to any mandatory statutory provisions on liability.**

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use. German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

**If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.**

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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