<b>DE</b> Wärmeunterbett Gebrauchsanweisung..... 2	<b>PL</b> Nakładka elektryczna na łóżko Instrukcja obsługi..... 51
<b>EN</b> Heated underblanket Instruction for Use ..... 9	<b>NL</b> Elektrische onderdeken Gebruiksaanwijzing..... 58
<b>FR</b> Matelas chauffant Mode d'emploi..... 16	<b>DA</b> Sengevarmer Betjeningsvejledning..... 65
<b>ES</b> Calientacamás eléctrico Instrucciones para el uso..... 23	<b>SV</b> Värmemadrass Brugsanvisning ..... 72
<b>IT</b> Coprimaterasso termico Istruzioni per l'uso ..... 30	<b>NO</b> Varmelaken Bruksveiledning ..... 79
<b>TR</b> Isıtmalı yatak altı Kullanma Talimatı..... 37	<b>FI</b> Lämpöpatja Käyttöohje..... 86
<b>RU</b> Электрическая простыня Инструкция по применению..... 43	

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# ENGLISH

**Read these instructions for use carefully. Observe the warnings and safety notes. Keep these instructions for use for future reference. Make the instructions for use accessible to other users. If the device is passed on, provide the instructions for use to the next user as well.**

## Contents

1. Items included in the package .....	10	4.4 Setting the temperature.....	13
1.1 Description .....	10	4.5 Switching off .....	13
2. Important instructions		5. Cleaning and care.....	14
Retain for future use .....	10	6. Storage.....	14
3. Intended use.....	12	7. Disposal.....	15
4. Operation.....	12	8. In the event of problems.....	15
4.1 Safety .....	12	9. Technical data .....	15
4.2 Putting into operation.....	13	10. Guarantee/service .....	15
4.3 Switching on.....	13		

## Explanation of symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	<b>WARNING:</b> Warning of risks of injury or health hazards		
	<b>CAUTION:</b> Safety information about possible damage to appliance/accessories		
	<b>NOTE:</b> Important information		
	Read the instructions		Do not insert pins
	Do not use folded or rucked		Not to be used by very young children (0 - 3 years)
	Manufacturer		
	The device has double protective insulation and therefore complies with protection class 2.		
	Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).		
	Separate the packaging elements and dispose of them in accordance with local regulations.		

	Marking to identify the packaging material. A = Material code, B = Material number: 1-7 = Plastics, 20-22 = Paper and cardboard		
	Separate the product and packaging elements and dispose of them in accordance with local regulations.		
	This product satisfies the requirements of the applicable European and national directives.		
	The products demonstrably meet the requirements of the technical Regulations of the EAEU.		
	Wash at a maximum temperature of 30 °C, Very gentle wash		Do not iron
			Do not bleach
	Do not dry-clean		Do not dry in a tumble dryer


## 1. Items included in the package

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- 1 Heated underblanket
- 1 Control
- 1 Instructions for use

### 1.1 Description

- 1. Power plug
- 2. Power cord
- 3. Control
- 4. Illuminated temperature settings
- 5. Slider for ON/OFF and temperature settings
- 6. Plug-in coupling

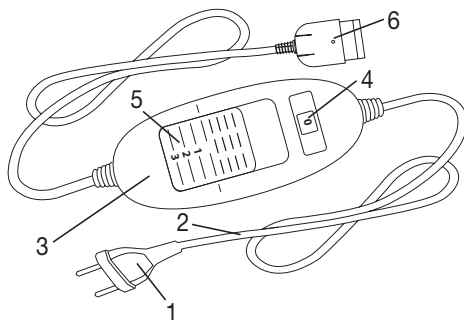## 2. Important instructions

### Retain for future use

#### WARNING

- Non-observance of the following notes may result in personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information is not only intended to protect your health and the health of others, it should also protect the product. For this reason, pay attention to these safety notes and include these instructions when handing over the product to others.

- This heated underblanket must not be used by persons who are not sensitive to heat or by other vulnerable persons who may not be able to react to overheating (e.g. diabetics, people with skin alterations due to illness or scarred tissue in the application area, after taking pain relief medication or alcohol).
- This heated underblanket must not be used by very young children (0-3 years old) as they are unable to respond to overheating.
- This heated underblanket can be used by children older than 3 and younger than 8 years of age provided they are supervised. For this, the control must always be set to the minimum temperature.
- This heated underblanket may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised and have been instructed on how to use the heated underblanket safely, and are fully aware of the consequent risks of use.
- Children must not play with the heated underblanket.
- Cleaning and user maintenance must not be performed by children unless supervised.
- This heated underblanket is not designed for use in hospitals.
- This heated underblanket is only intended for domestic/private use, not for commercial use.
- Do not insert needles.
- Do not use when folded or bunched up.
- Do not use if wet.
- Before using it on an adjustable bed, check that the heated underblanket and the cables are not, for example, trapped in a hinge or bunched up.
- This heated underblanket may only be used in conjunction with the control specified on the label.
- This heated underblanket must only be connected to the mains voltage that is specified on the label.
- The electrical and magnetic fields emitted by this heated underblanket may interfere with the function of a pacemaker. However, they are still well below the limits: electrical field strength: max. 5000 V/m, magnetic field strength: max. 80 A/m, magnetic flux density: max. 0.1 millitesla. Please therefore consult your doctor and the manufacturer of your pacemaker before using this heated underblanket.
- Do not pull, twist or make sharp bends in the cables.
- If they are not positioned properly, there may be a risk of becoming entangled in, being strangled by, tripping over, or stepping on the cable and control of the heated underblanket. The user must ensure that excess lengths of cable, and cables in general, are safely routed.

- Please check this heated underblanket frequently for signs of wear and tear or damage. If any such signs are evident, if the heated underblanket has been used incorrectly or if it no longer heats up, it must be checked by the manufacturer before being switched on again.
- Under no circumstances should you open or repair the heated underblanket (including the accessories) yourself because faultless functionality can no longer be guaranteed thereafter. Failure to observe this will invalidate the guarantee.
- If the mains connection cable of this heated underblanket is damaged, it must be disposed of. If it cannot be removed, the heated underblanket must be disposed of.
- Do not expose control and cables to direct sunlight.
- When this heated underblanket is switched on:
  - Do not place any sharp objects on it
  - Do not place any heat sources, such as hot water bottles, heat pads or similar, on it
- The electronic components in the control warm up when the heated underblanket is in use. For this reason, the control must never be covered or placed on the heated underblanket when it is in use.
- It is essential to observe the information relating to the following chapters: Operation, Cleaning and maintenance, Storage.
- If you should have any questions about using our devices, please contact our Customer Services department.

### 3. Intended use

#### CAUTION

This heated underblanket is designed for warming beds, only.

## 4. Operation

### 4.1 Safety


#### CAUTION

- This heated underblanket is equipped with a SAFETY SYSTEM. This sensor technology prevents the entire surface of the heated underblanket from overheating by automatically switching off the heated underblanket in the event of a fault. If the safety system has switched off the heated underblanket, the temperature setting is not illuminated any more even when the heated underblanket is switched on.
- Please note that the heated underblanket is not operable any more due to safety reasons in this case and has to be sent to the service address indicated.
- Do not connect the defective heated underblanket with another control of the same type. This would trigger a permanent switch-off via the control's safety system.

## 4.2 Putting into operation

This heated underblanket must be fixed onto the mattress.

1. Leave space for the pillow and place the heated underblanket on the mattress with the plug-in connector facing the pillow. Here, the plug-in connector is located on the underside of the heated underblanket.
2. Stretch all of the elastic straps over the corners of the mattress. All 4 corners of the heated underblanket are then fixed onto the mattress.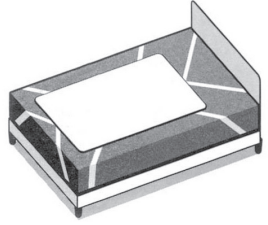

### CAUTION

Regularly check the position of the heated underblanket on the mattress. The heated underblanket must never be used if it is folded.


After you have fixed the heated underblanket to the mattress, simply put your bed sheet on top as usual so that the heated underblanket is between the mattress and the bed sheet.

### CAUTION

Make sure that the heated underblanket is spread out entirely flat and that it will not rumple up or crease when in use.

- To operate the heated underblanket connect the control to the heated underblanket by plugging in the connector.
- Then plug in the power plug into the mains outlet.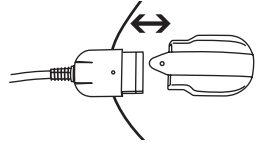

## 4.3 Switching on

To switch on the heated underblanket, set the slider for ON/OFF and temperature settings to setting 1, 2 or 3. The temperature settings are illuminated when switched on.

## 4.4 Setting the temperature

Setting 0: OFF

Setting 1: minimum heat

Setting 2: medium heat

Setting 3: maximum heat

### NOTE:

- The fastest way to warm up the heated underblanket is to initially set the highest temperature setting.
- We strongly recommend switching on the heated underblanket approx. 30 minutes before going to bed and covering it with the quilt to prevent the warmth from escaping.

### WARNING

If the heated underblanket is being used over several hours, we recommend that you set the lowest temperature setting on the control to avoid the body overheating and thereby any potential burns to the skin or heat stroke.

## 4.5 Switching off

To switch off the heated underblanket, set the slider for ON/OFF and temperature settings to the "0" (OFF) position. The temperature settings are then no longer illuminated.

**NOTE:**

If the heated underblanket is not used for several days, set the slider for ON/OFF and temperature settings to the "0" (OFF) position and remove the power plug from the socket. Then remove the control from the heated underblanket by disconnecting it at the connector.

## 5. Cleaning and care

**WARNING**

Always remove the power plug from the mains outlet and remove the control from the heated underblanket by disconnecting it at the connector. Otherwise there is a risk of electric shock.

**CAUTION**

The control must never get in contact with water or other liquids. Otherwise the control may be damaged.

- To clean the control, use a dry, lint-free cloth. Do not use any chemical or abrasive cleaning agents.
- The textile cover can be cleaned in accordance with the symbols on the label and must be removed from the heating pad prior to cleaning.

**CAUTION**

Note that the heated underblanket must not be dry-cleaned, wrung out, machine dried or ironed.

- The heated underblanket is machine washable.
- Set the washing machine to a particularly gentle, 30 °C cycle (woolens programme). Use a mild detergent with dosage according to the manufacturer's instructions.

**CAUTION**

Note that the heated underblanket will suffer from too frequent washing. For its entire life, therefore, the heated underblanket should be washed not more than 5 times in a washing machine.

- While the heated underblanket is still damp after washing, shape it to its original form and size and place it flat over a clothes drying rack to dry.

**CAUTION**

- Do not attach the heated underblanket to washing lines with washing pegs or the like. Otherwise the heated underblanket may be damaged.
- Only reconnect the control to the heated underblanket after the connector and the heated underblanket are completely dry. Otherwise the heated underblanket may be damaged.

**WARNING**

Under no circumstances switch on the heated underblanket to dry it. Otherwise there is a risk of an electric shock.

## 6. Storage

When not in use we recommend to store the heated underblanket in its original packaging. For this purpose, disconnect the control from the heated underblanket by unplugging the plug-in coupling.

**CAUTION**

- Allow the heated underblanket to cool down before folding. Otherwise the heated underblanket may be damaged.
- Do not crease the heated underblanket by placing items on top of it during storage.

## 7. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

## 8. In the event of problems

Problem	Cause	Solution
The temperature settings are not illuminated while - the control is properly connected to the heating underblanket - the power plug is connected to a working socket - the slider is set to 1, 2 or 3	The safety system has switched the heating underblanket off permanently.	Send the heating underblanket and the control for servicing.

## 9. Technical data

For technical data, see the rating label on the heated underblanket.

## 10. Guarantee/service

Further information on the guarantee and guarantee conditions can be found in the guarantee leaflet supplied.

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