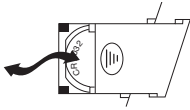
- (D)** Glaswaage
Gebrauchsanleitung
- (GB)** Glass scale
Instruction for Use
- (F)** Pèse-personne en verre
Mode d'emploi
- (E)** Báscula de vidrio para personas
Instrucciones para el uso
- (I)** Bilancia in vetro
Istruzioni per l'uso
- (TR)** Cam Terazı
Kullanma Talimatı
- (RUS)** Стеклянные весы
Инструкция по применению
- (PL)** Waga szklana
Instrukcja obsługi
- (NL)** Weegschaal met glazen
weegplateau
Gebruikershandleiding
- (P)** Balança de vidro
Instruções de utilização
- (GR)** Γυάλινη ζυγαρία
Οδηγίες χρήσης
- (DK)** Glasvægt
Betjeningsvejledning
- (S)** Glasvåg
Bruksanvisning
- (N)** Glassvekt
Bruksveiledning
- (FIN)** Lasivaaka
Käyttöohje
- (CZ)** Skleněná váha
Návod k použití
- (SLO)** Steklena tehtnica
Navodila za uporabo
- (H)** Üvegmérleg
Használati utasítás
- (RO)** Cântar de sticlă
Instrucțiuni de utilizare
- الميزان الزجاجي
دليل الاستعمال

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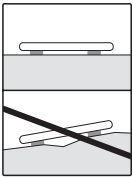
- (GB)** Getting started
- (F)** Mise en service
- (E)** Puesta en marcha
- (I)** Messa in funzione
- (TR)** İlk çalıştırma

- (D)** Batterie einlegen oder Isolierschutzstreifen entfernen.
- (GB)** Insert batteries or remove insulating strip.
- (F)** Insérez les piles ou retirez la bande de protection isolante.
- (E)** Colocar las pilas o quitar las tiras de protección de aislamiento.

- (D)** Waage auf einen ebenen und festen Boden stellen.
- (GB)** Place the scale on a secure, flat surface.
- (F)** Posez la balance sur un sol plat et dur.

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GB General advice

- Max. capacity 180 kg (396 lb / 28 st), Graduation 100 g (0.2 lb / 1 lb).
- Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required. Never immerse the scale in water or rinse it under running water.
- Protect the scale against knocks, damp, dust, chemicals, drastic changes in temperature, electromagnetic fields and sources of heat that are too near.
- Never step onto the outer edge of the scale with one foot as this could cause the scales to tip!
- Keep packaging material away from children!
- Not intended for commercial use.
- Should you have any questions regarding operation and usage of our appliances, please contact your local retailers or Customer Service.
- Before making a claim, please check the batteries and replace them if necessary.
- Repairs must only be carried out by Customer Services or authorised suppliers.
- Spent batteries and rechargeable batteries do not constitute normal household waste! They are considered to be toxic waste and, as such, should be disposed of in special containers, toxic waste collection points or brought to electrical goods dealers.

You are legally obliged to dispose of the batteries.

- Note: Batteries containing pollutant substances are marked as follows:

Pb= Battery contains lead,
Cd = Battery contains cadmium,
Hg = Battery contains mercury.

- Follow local regulations regarding the disposal of materials. The appliance

should be disposed of according to Regulation 2002/96/EC-WEEE (Waste Electrical and Electronic Equipment). In case of queries, contact the municipal authorities responsible for waste disposal in your area.

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